

# LaserSkinRenewal

- Reduce wrinkles
- Treat brown spots
- Improve skin irregularities
- Smooth and treat uneven pigmentation
- Treat dark undereye circles
- Reduce pore size<sup>1</sup>
- Rejuvenate hands and neckline

## FOOTNOTES

1. Through tissue coagulation

Cynosure is a registered trademark of Cynosure, Inc.  
SmartSkin is a trademark of Cynosure. LaserSkinRenewal is a servicemark of Cynosure, Inc. 921-0172-000 Rev.2

SmartSkin CO<sub>2</sub>  
LaserSkinRenewal<sup>SM</sup>

Rejuvenate  
and Restore  
Your Skin

SmartSkin™ CO<sub>2</sub>

Reduce wrinkles

Treat brown spots

Improve skin  
irregularities

## Restore Your Skin Through Laser Skin Rejuvenation

Over the years signs of aging appear as a result of exposure to the sun (photoaging) and the effects of gravity and natural aging.

The texture, tone and pigmentation of your skin can be affected, often resulting in unwanted wrinkles and fine lines, sun spots and larger pores. This look of premature aging can negatively impact how we feel about our appearance and how we are perceived by others.

The good news is that new advancements in laser technology provide a means of rejuvenating and restoring your skin to a more youthful, smoother look.

SmartSkin laser fractional resurfacing, the latest advancement in laser skin renewal, optimizes your treatment outcomes while minimizing your recovery time.

Ask your doctor how you can benefit from a SmartSkin Laser Rejuvenation treatment.

### How does it work?

Your doctor will use the SmartSkin laser to apply a series of tiny pulses of laser energy to your skin. As these microscopic holes heal, new younger looking skin is revealed and collagen is stimulated deep within your skin.

### How many treatments will I need?

You and your doctor will carefully determine the best course of treatment based on your cosmetic and downtime expectations. You may opt for two or three sessions with minimal downtime. Or if you're able to take a longer downtime, experience dramatic results in just one treatment!

### What does the procedure feel like?

A topical anesthetic is generally applied before the procedure, although your physician may recommend other approaches to make you comfortable. You will relax in the doctor's chair and feel heat and tingling as the laser scanner moves across your face.

### How long will the results last?

The entire face can be treated in as little as 15-30 minutes, with results lasting years, with proper skin care and protection.

### What should I expect following treatment?


Immediately after the treatment, your skin will feel warm and appear to be sunburned. Your body naturally starts working by replacing the aged skin cells with new ones. After a few days, you will notice a more even skin tone, smoother

## Unmatched Results with SmartSkin CO<sub>2</sub>

**B** Before **A** After



Courtesy of Deborah S. Sarnoff, M.D.

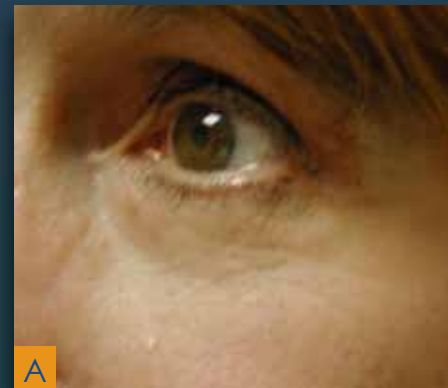

Courtesy of Bruce Katz, M.D.



Courtesy of Deborah S. Sarnoff, M.D.

skin texture, and healthier-looking skin. You may see some redness and swelling which is completely normal. Over time, collagen will regenerate and restore the supporting structure of your skin. You may be asked to avoid sun exposure for a period of time.

### What is the recovery time?

Because SmartSkin CO<sub>2</sub> has the distinct ability to deliver micro pulses of energy, you will experience a significantly shorter recovery time than with fully ablative lasers. Each tiny hole will quickly heal and in 2-7 days, you will be able to apply makeup on the treated surface and return to your normal lifestyle. You and your doctor may choose a more aggressive treatment regimen for more significant results with a downtime of three to five days with just one treatment.

"Over the past five years, I noticed an uneven skin tone on my face, including brown spots and wrinkling around my eyes. Every day I would cover these blemishes with makeup. I had one SmartSkin laser treatment to my face. My eyes look great now, and it has dramatically improved my appearance, including the tone and the texture of my skin. Everyone is telling me that my skin looks great! I'm thrilled with the results!"

— Tamara G.